




Hamilton G5 Rear Panel-Metal Shield Disassembly

How to remove the rear cover and the metal shield of the interaction panel.

Written By: Dane Dormedy

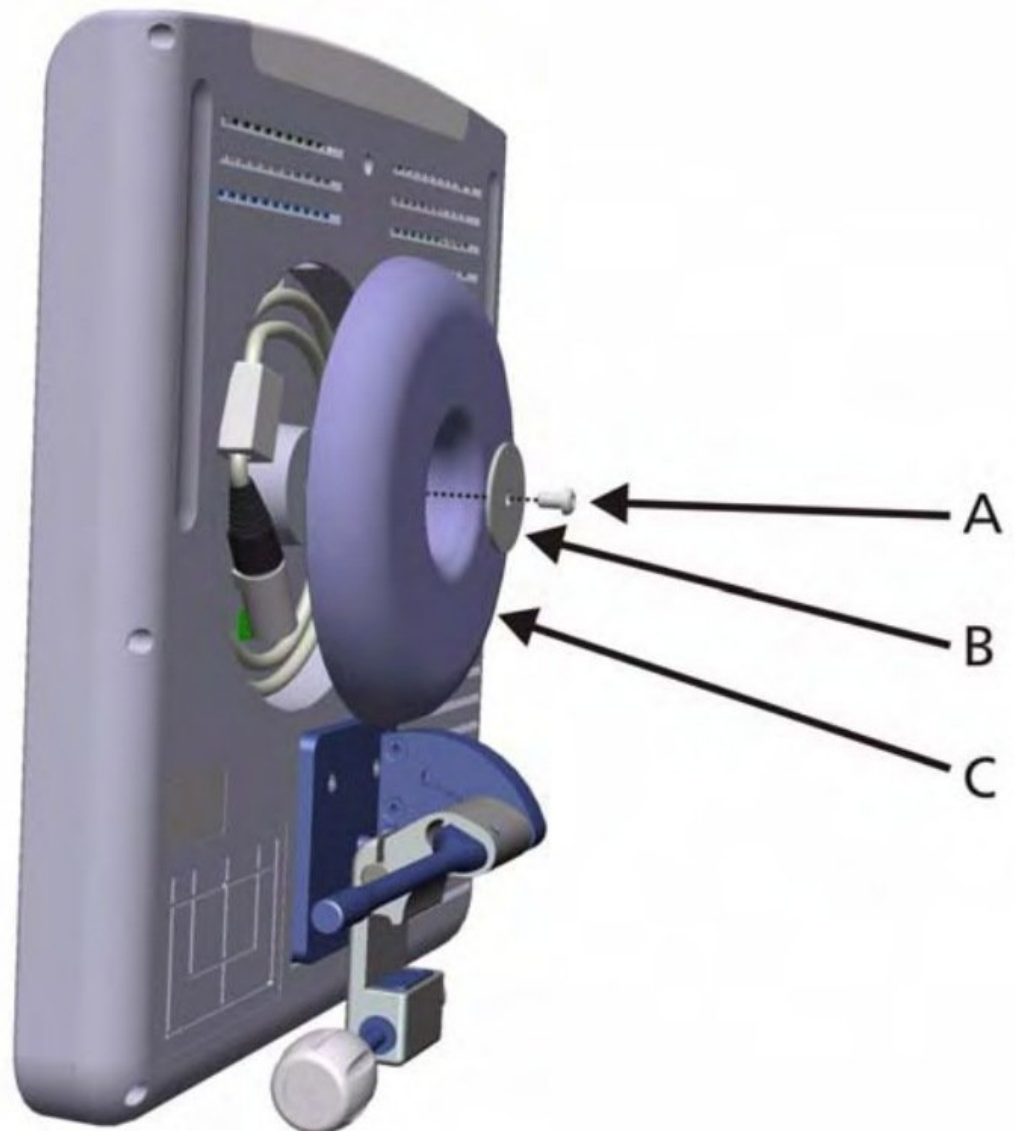

INTRODUCTION


This guide will show you how to remove and replace the rear cover and the metal shield of the Hamilton G5 ventilator. For more information on replacing the rear cover and metal shield, please check section 9-24 in the [service manual](#).



TOOLS:


- [Hex Allen Key](#) (1)
-

Step 1 — Hamilton G5 Rear Panel-Metal Shield Disassembly

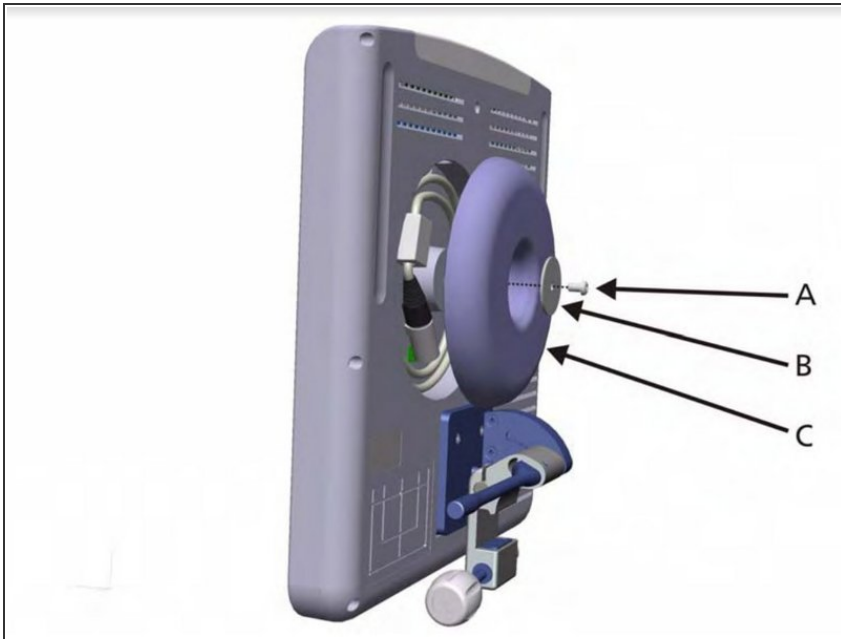


- Remove the screw (A) and washer(B) and remove the cable management cover (C).

Step 2

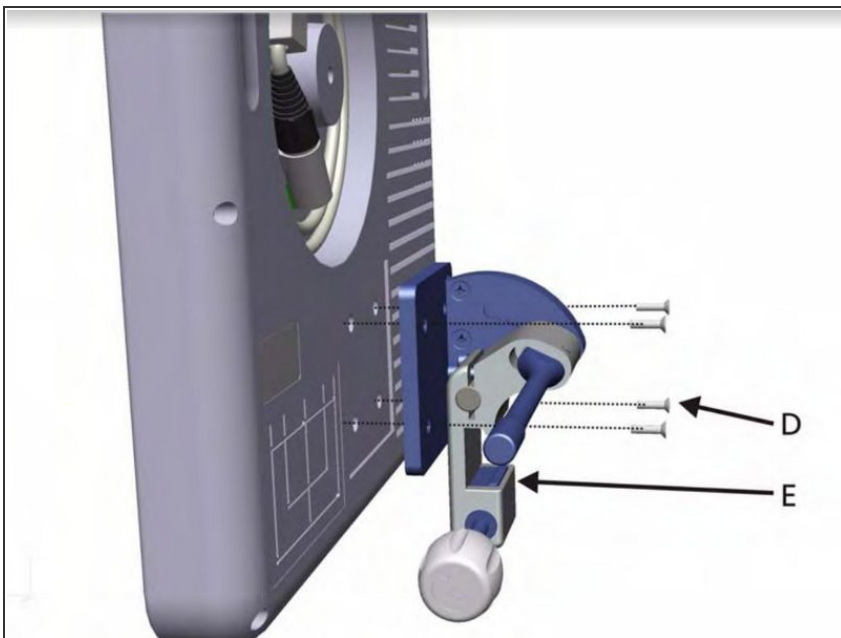


- Remove the 4 screws (D) to remove the panel's swivel mount (E).

Step 3

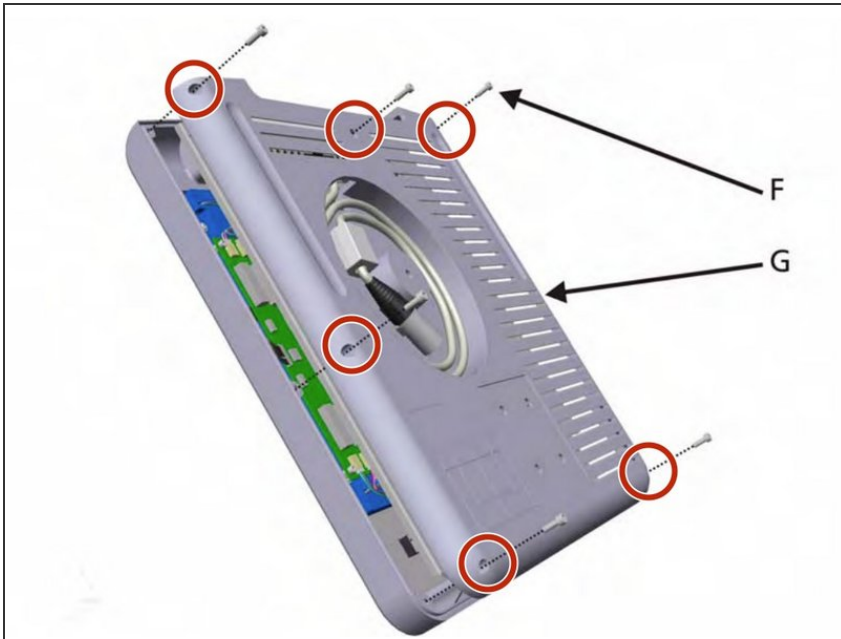

- Remove the 6 screws (F) and remove the back panel (G).

Step 4

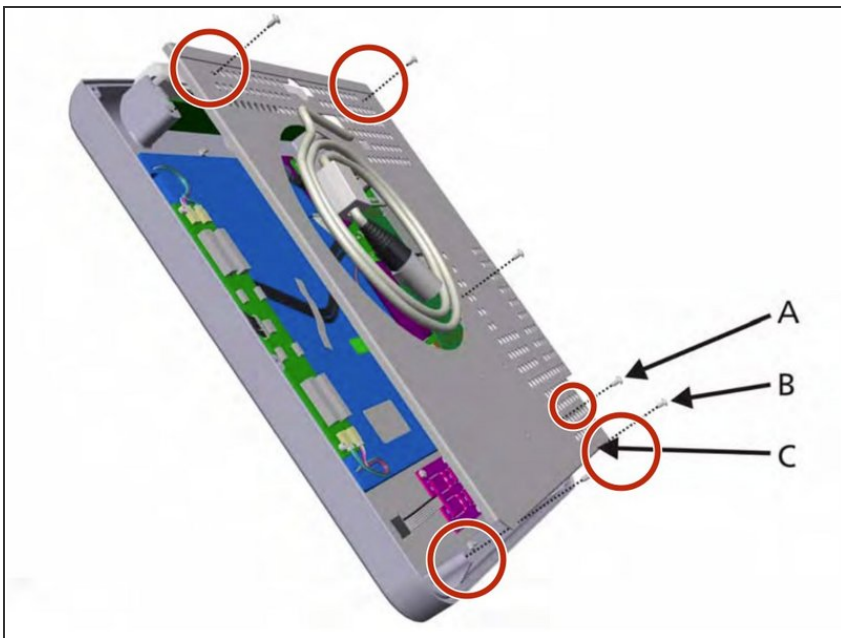

- Remove the 5 screws and remove the metal shield.

To reassemble your device, follow these instructions in reverse order.