


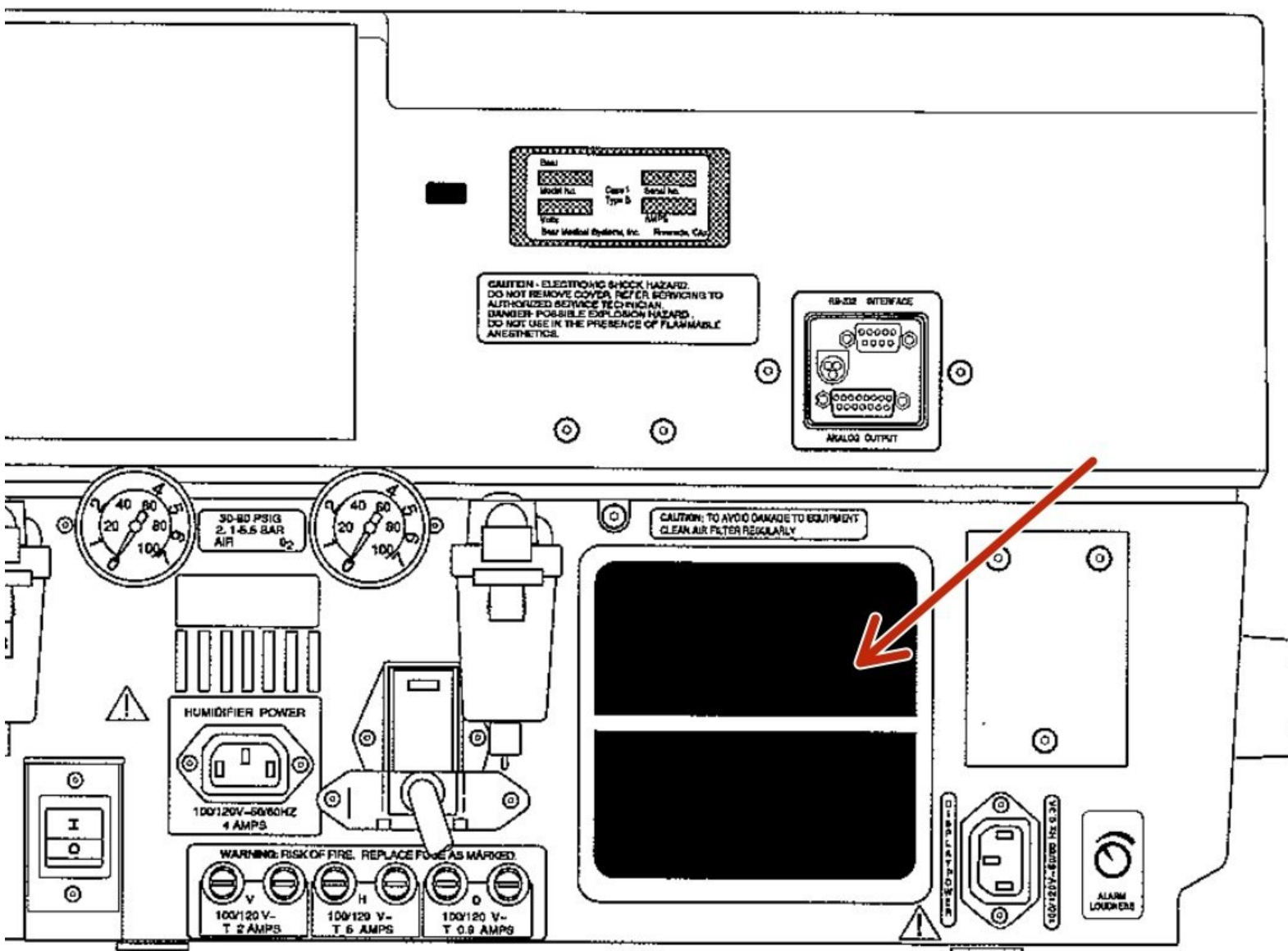

# Bear 1000 Filter Replacement

Use this guide to replace the ambient air filters in your Bear 1000 ventilator.

Written By: Eli Ferman



---

## INTRODUCTION

This guide will walk you through the process of replacing the ambient air intake filters in a Bear 1000 Ventilator. Found on page 97 in the full service manual.



### TOOLS:


- [Flathead Screwdriver](#) (1)
-

## Step 1 — Filter



- Turn off the ventilator and disconnect from AC power and pressurized gas at the wall sources.

## Step 2

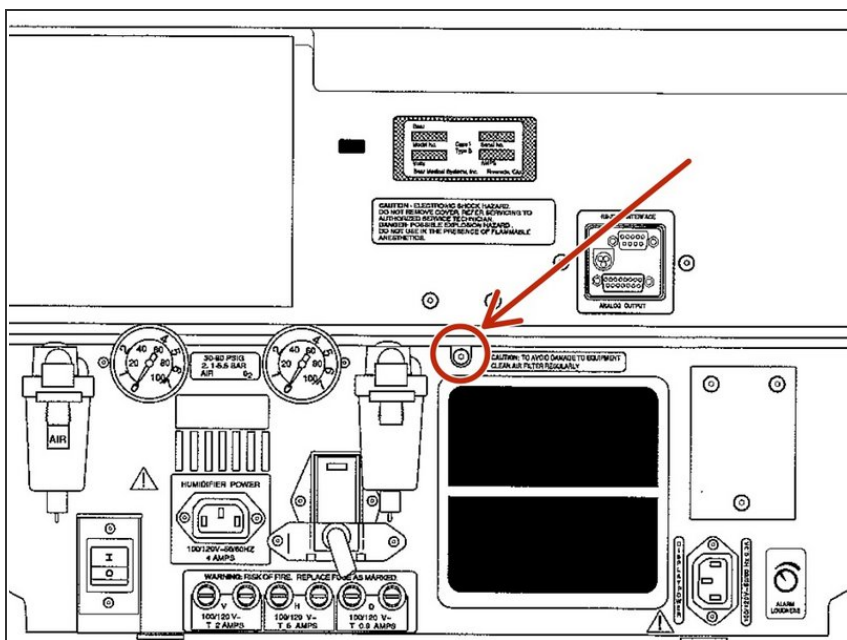

- Open the ventilator by removing the console retaining screw.

### Step 3



- Vacuum all surfaces inside the ventilator. Be sure to reach areas around connectors, circuit boards and pneumatic assemblies.

### Step 4



- Remove old filters in back panel and replace with new ones.

To reassemble your device, follow these instructions in reverse order.