



# How to Tighten GEN 2 Wheelchair Wheels

If GEN 2 wheelchair wheels are crooked, wobbly or unstable, then the wheels need to be tightened.

Written By: Kali Slaughter



## INTRODUCTION

The rear wheels of the wheelchair are used for propelling. This guide will show users how to tighten the main wheels on the GEN 2 wheelchair. The process is simple and only requires one tool.



### TOOLS:

- [Socket 19mm](#) (1)
  - [19 mm wrench](#) (1)
-

## Step 1 — How to Tighten GEN 2 Wheelchair Wheels



- Inspect the chair to see which wheel is loose.
- Tighten the outer bolt with a 19 mm socket wrench.

## Step 2



- Tighten the nut facing the inside of the chair with a 19 mm wrench.

If wheel does not feel properly secured, keep tightening the wheel in small amounts until the wheel is secure.

